# NEST S061AA00

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## ⚠ WARNING / ATTENTION

Activities involving the use of this equipment are inherently dangerous. You are responsible for your own actions and decisions.

- Before using this equipment, you must:
- Read and understand all instructions for Use.
  - Get specific training in its proper use.
  - Become acquainted with its capabilities and limitations.
  - Understand and accept the risks involved.

**FAILURE TO HEED ANY OF THESE WARNINGS MAY RESULT IN SEVERE INJURY OR DEATH.**

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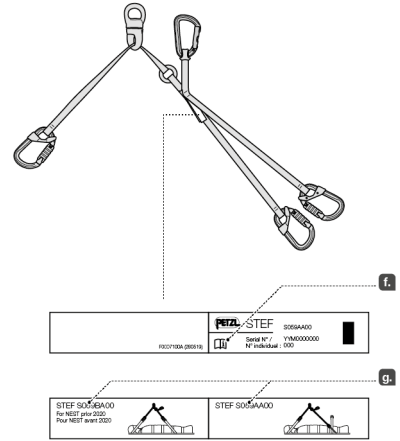
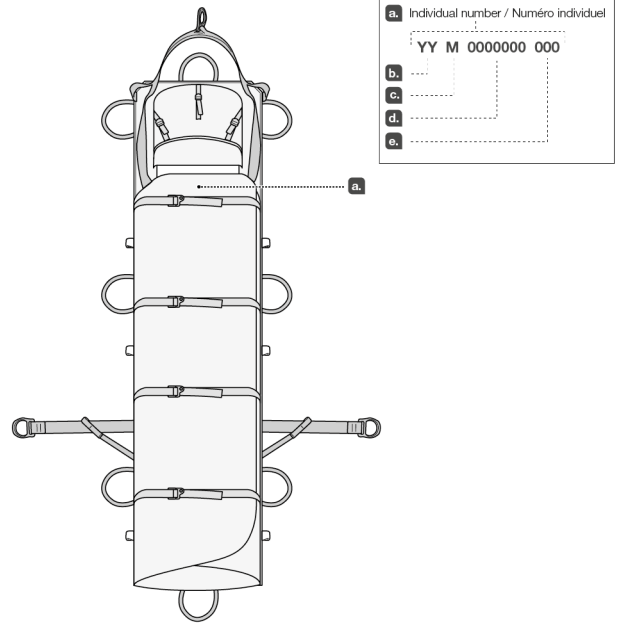
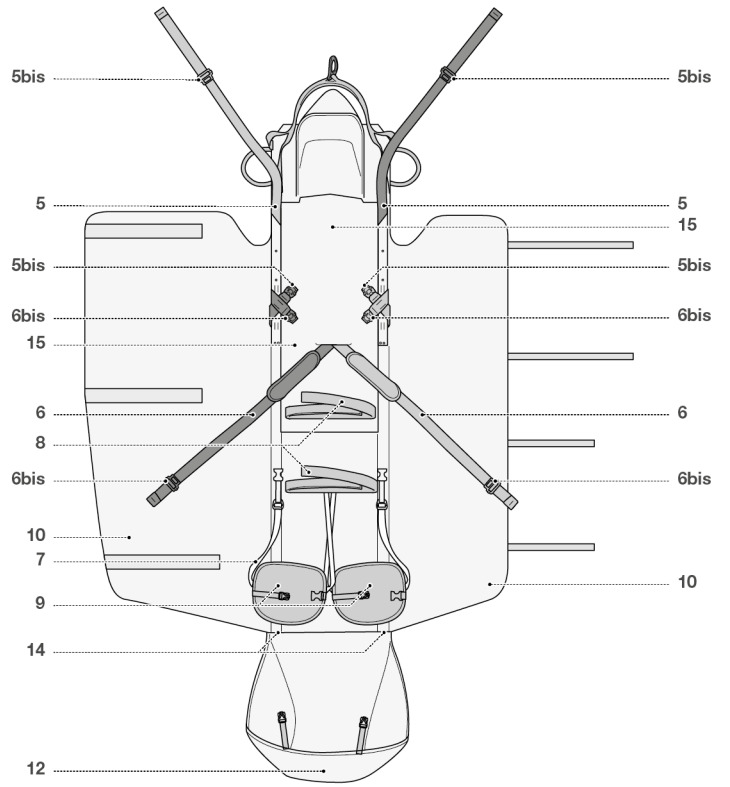
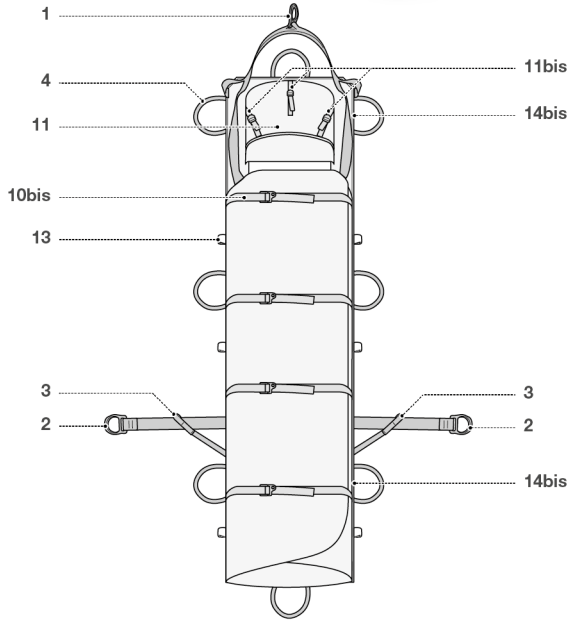
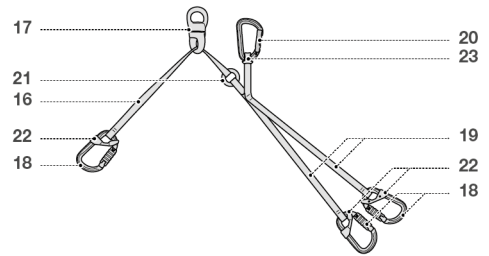
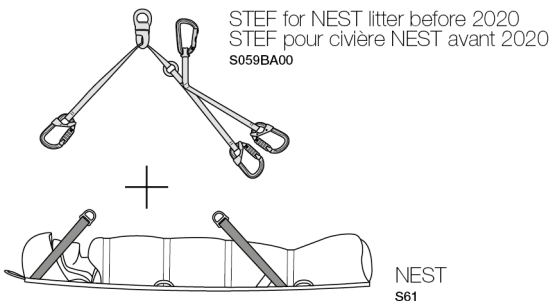
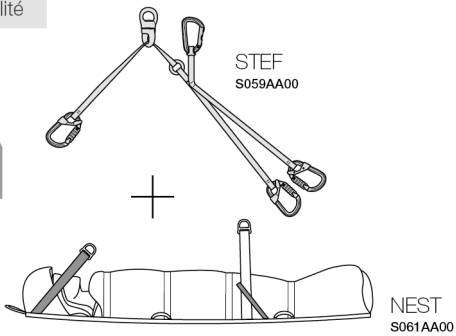
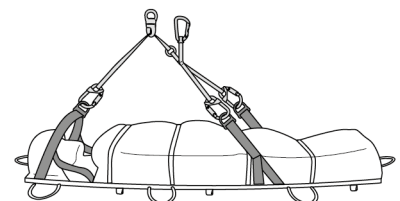
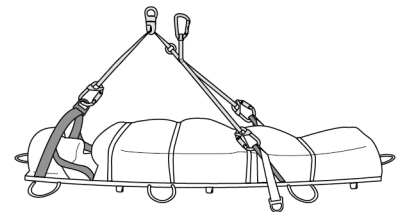
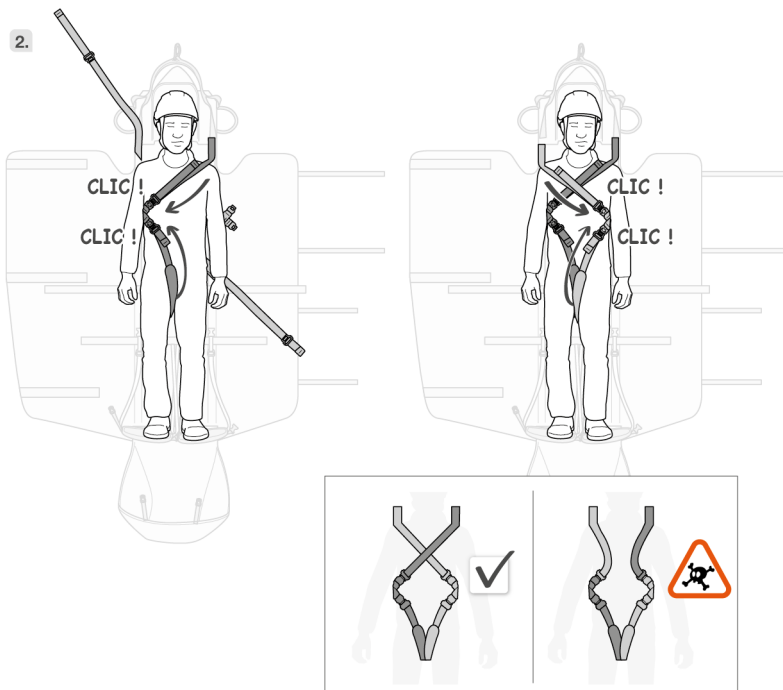
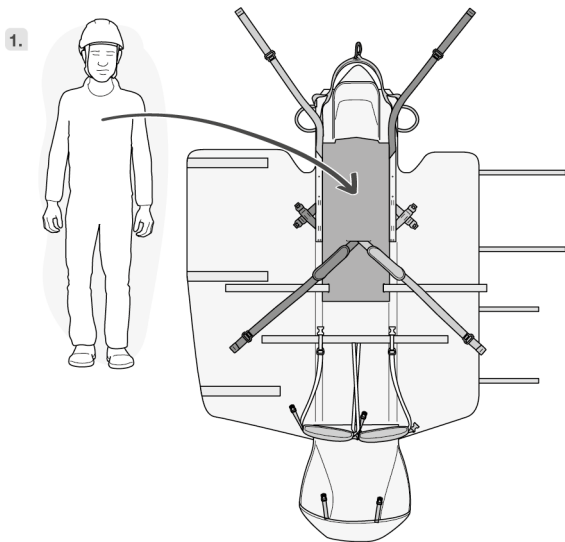
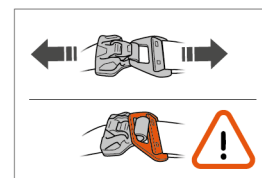
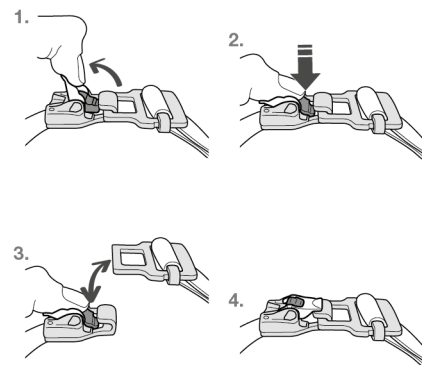
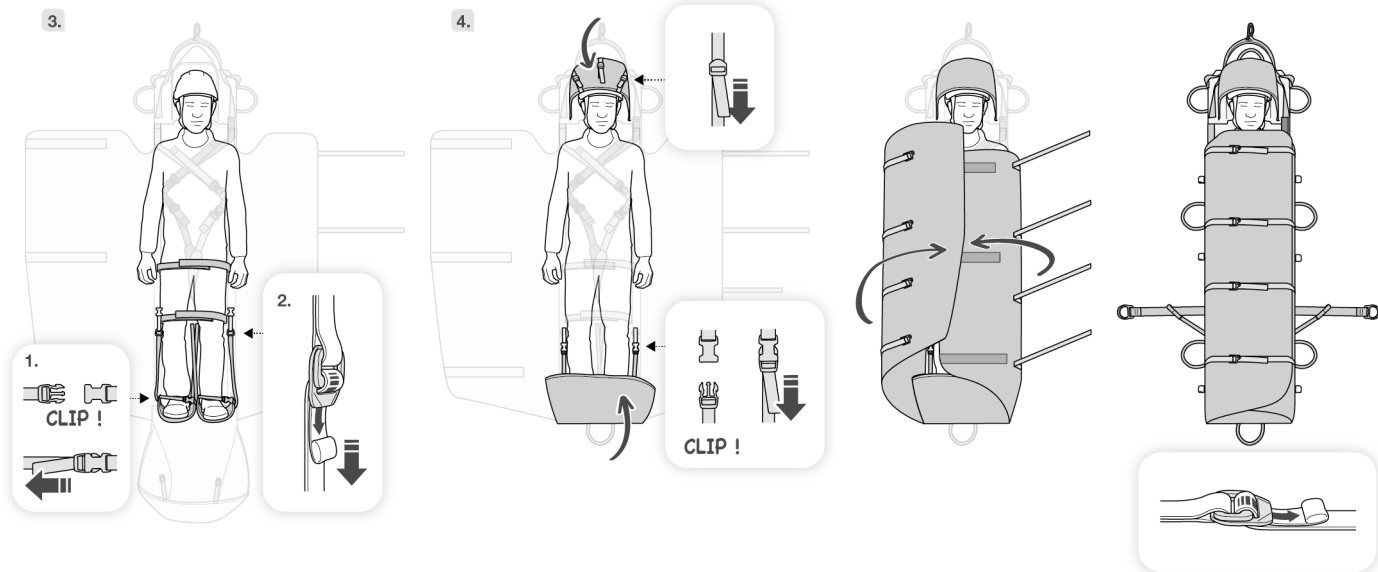
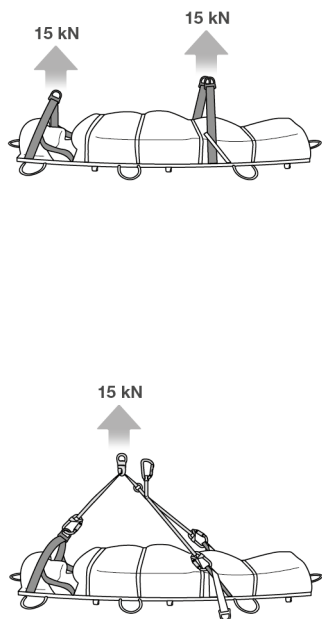
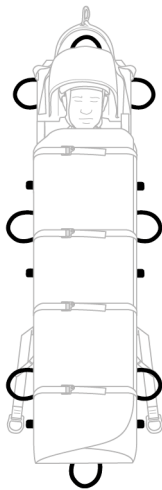
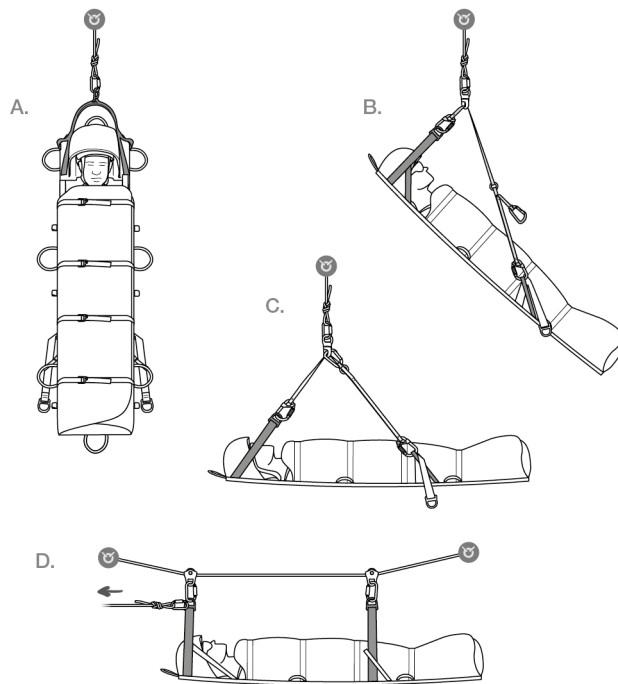
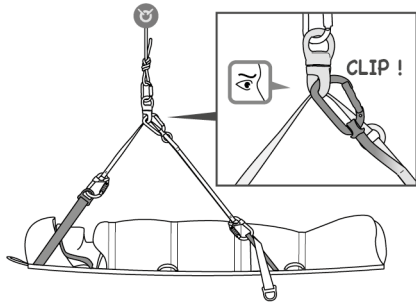
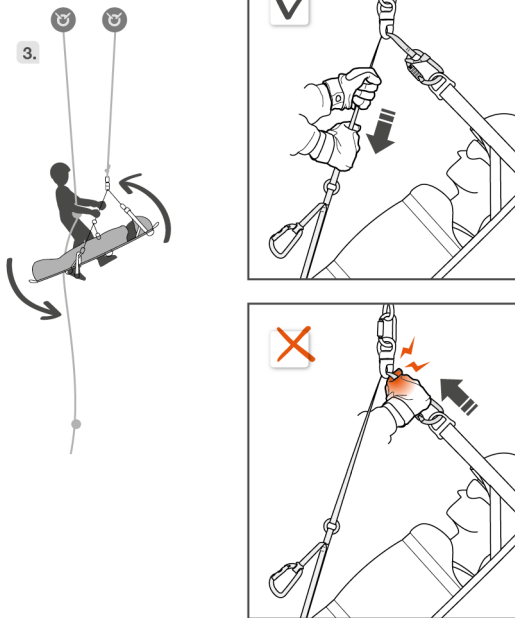
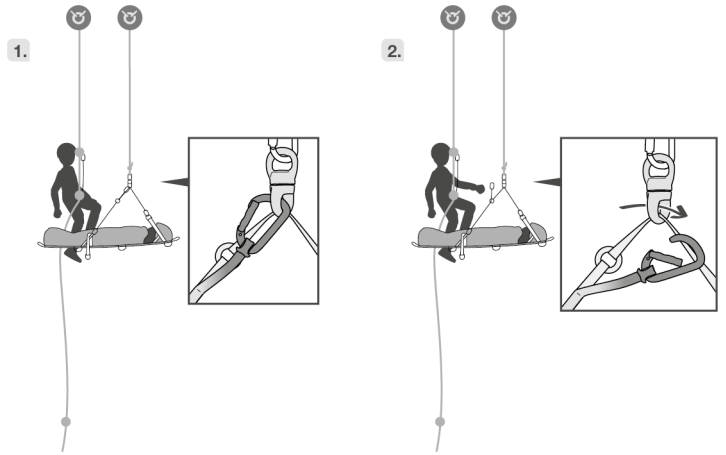
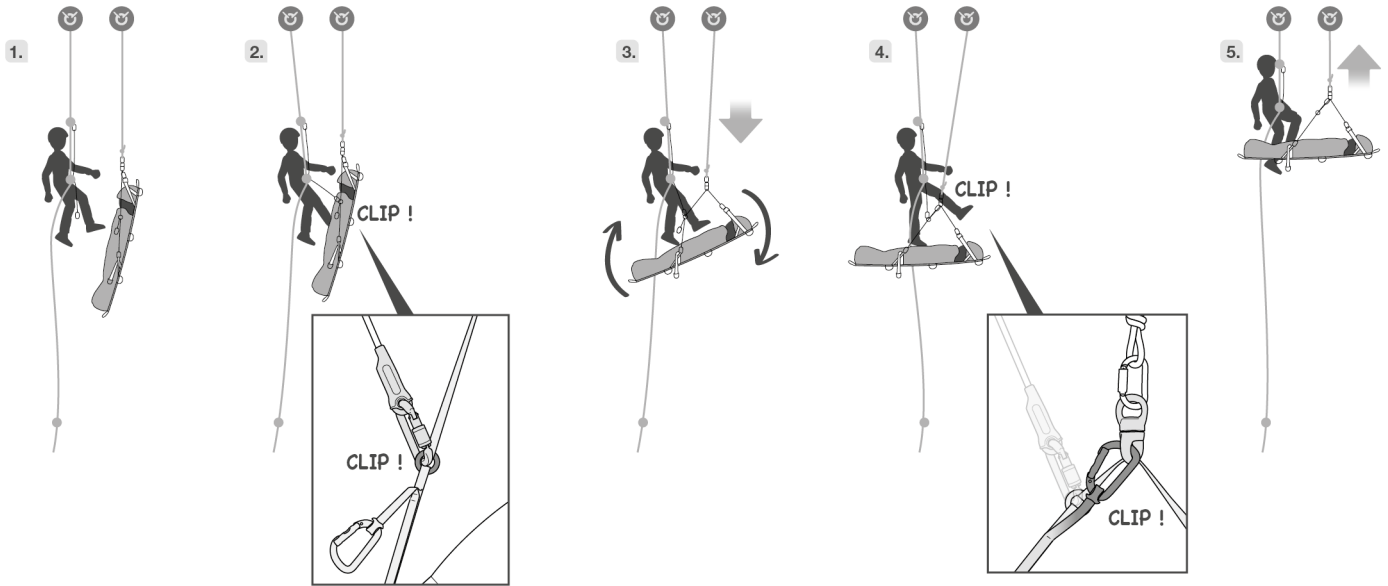
**10c.** Changing from inclined position to horizontal position  
Transfert position quasi verticale à horizontale

**11.** Transporting the litter  
Transport de la civière

Accessories  
Accessoire

Bag for litter  
Sac pour civière  
S062AA00

**12.** Additional information  
Informations complémentaires

<p><b>A. Lifetime / Durée de vie</b></p> <p>Serial n / N° de série 00 XXX XX XXXX + 10 years ans</p>	<p><b>B. Markings / Marquage</b></p>	<p><b>C. Acceptable T° / T° tolérées</b></p> <p>+ 50°C / + 122°F - 30°C / - 22°F</p>
<p><b>D. Precautions for use / Précautions d'usage</b></p> <p>etc...</p>		
<p><b>E. Cleaning - Disinfection / Nettoyage - Désinfection</b></p> <p>+ 30°C maxi. + 86°F maxi.</p>		
<p><b>F. Drying / Séchage</b></p> <p>+ 30°C maxi. + 86°F maxi.</p>	<p><b>G. Storage - Transport / Stockage - transport</b></p>	
<p><b>H. Modifications - Repairs / Modifications - Réparations</b></p> <p>→ Petzl</p>	<p><b>I. FAQ - Contact / Questions - Contact</b></p> <p>→ petzl.com</p>	

These instructions explain how to correctly use your equipment. Only certain techniques and uses are described.

The warning symbols inform you of some potential dangers related to the use of your equipment, but it is impossible to describe them all. Check Petzl.com for updates and additional information.

You are responsible for heeding each warning and using your equipment correctly. Any misuse of this equipment will create additional dangers. Contact Petzl if you have any doubts or difficulty understanding these instructions.

## 1. Field of application

NEST: litter (200 x 50 x 5 cm) for rescue in difficult access situations. Designed for a weight of 150 kg maximum and height between 1.50 m and 1.90 m.

STEF: orientation device for the NEST litter. For use only with the NEST litter.

STEF for pre-2020 NEST litter: orientation device for pre-2020 NEST litter. For use only with the pre-2020 NEST litter.

These products must not be loaded beyond their strength ratings, nor be used for any purpose other than that for which they are designed.

## Responsibility

### WARNING

**Activities involving the use of this equipment are inherently dangerous. You are responsible for your own actions and decisions**

Before using this equipment, you must:

- Read and understand all Instructions for Use.
- Get specific training in its proper use.
- Become acquainted with its capabilities and limitations.
- Understand and accept the risks involved.

**Failure to heed any of these warnings may result in severe injury or death.**

WARNING: training is essential before use. This training must be suitable for the practices defined in the Field of Application.

This product must only be used by competent and responsible persons, or those placed under the direct and visual control of a competent and responsible person. You are responsible for your actions, your decisions and your safety and you assume the consequences of same. If you are not able, or not in a position to assume this responsibility, or if you do not fully understand the Instructions for Use, do not use this equipment.

## 2. Nomenclature

### NEST

(1) Head end attachment point, (2) Side attachment points, (3) STEF attachment points, (4) Carry handles, (5) Torso straps, (6 bis) FAST LT PLUS torso strap buckles, (6) Leg straps, (6 bis) FAST LT PLUS leg strap buckles, (7) Foot support adjustment straps, (8) Positioning straps, (9) Foot supports, (10) Flaps, (10 bis) Flap closure straps, (11) Head cover, (11 bis) Head cover adjustment strap, (12) Foot cover, (13) Accessory attachment points, (14) Sleeve for longitudinal stays, (14 bis) Sleeve for transverse stays, (15) Foam pad.

### STEF

(16) Head end single strap, (17) SWIVEL, (18) Am'D, (19) Foot end double straps, (20) Sm'D, (21) Ring, (22) CAPTIV, (23) STRING.

### Principal materials

Straps: polyester.  
Buckles: steel, aluminum alloy.  
Safety attachment point: aluminum alloy.  
Tarp fabric: thermoplastic polyurethane (TPU).

## 3. Inspection, points to verify

Your safety is related to the integrity of your equipment.

Petzl recommends a detailed inspection by a competent person at least once every 12 months (depending on current regulations in your country, and your conditions of usage). Follow the procedures described at Petzl.com. Record the results on your PPE inspection form: type, model, manufacturer contact info, serial number or individual number, dates: manufacture, purchase, first use, next periodic inspection; problems, comments, inspector's name and signature.

### Before each use

Check the condition of the harness straps and attachment points.

Verify that the FAST LT PLUS buckles function properly.

Check the presence and condition of the STRINGS and the CAPTIVs, and that they are correctly assembled with the connectors / webbing.

Look for cuts, wear and damage due to use, to heat, to chemicals... Be particularly careful to check for cut or loose threads.

Consult the details of the inspection procedure to be carried out for each item of PPE at Petzl.com.

Contact Petzl if you have any doubts.

### During use

It is important to regularly monitor the condition of the product and its connections to the other equipment in the system. Make sure that all items of equipment are correctly positioned with respect to each other.

## 4. Compatibility

Verify that this product is compatible with the other elements of the system in your application (compatible = good functional interaction).

The STEF (S059AA00) is compatible with the 2020 NEST (S061AA00), but is not compatible with the pre-2020 NEST (S61).

## 5. Installing the STEF on the litter

The STEF single strap is connected to the NEST head end attachment point and the double straps are connected to the STEF attachment points on the NEST. The Sm'D carabiner must be placed upward. The straps must not be twisted.

Litter hauling is done only on the SWIVEL, or on the litter's head end attachment point.

Warning: depending on the products used, (configuration before 2020 or after 2020), be sure to use the correct attachment points by referring to the drawings.

## 6. Installing the patient

1. Place the litter on a stable base.

Open the flaps and lay them aside. Unbuckle the harness straps. Loosen the foot support straps.

Place the patient on the litter. The patient's shoulders are to be placed at the top of the foam pad, regardless of his or her height. The patient must be wearing a helmet and protective eyewear.

2. Lock the FAST LT PLUS buckles on the torso and leg straps, observing the color code. Warning: observe the correct strap configuration on the patient, as indicated in the drawings.

3. Fasten the positioning straps.

Adjust the length of the foot support straps and fasten the foot support buckles. Make sure all of the straps are tight enough to hold the patient securely, without creating painful pressure points.

4. Fold the head and foot covers over and fasten the head and foot cover buckles. Close the flaps, or fold them to the sides, depending on the temperature, presence of water... Tighten the flap straps. The patient's arms can be inside or outside of the litter.

Some safety elements may or may not be secured in place, depending upon the nature of the patient's wounds as determined by the physician, and the difficulty of the evacuation.

## 7. Attachment points

Breaking strength of the attachment points: 15 kN.

Use only these points to connect the litter to an anchor.

When using the side attachment points, be sure to always use the two points together.

## 8. Litter handling points

The litter handling points consist of the eight carry handles, as well as the eight accessory attachment points.

Handle breaking strength: 4 kN.

Accessory attachment point breaking strength: 1 kN.

Warning: these points must not be used to attach the litter to an anchor, nor to anchor a person.

## 9. Evacuation configurations

Each of the evacuation configurations applies to the following two configurations:

- As of 2020: the NEST on its own, or combined with the STEF.
- Before 2020: the pre-2020 NEST on its own, or combined with the STEF for the pre-2020 NEST.

Be sure to follow the diagram for installing the STEF on the litter for each of the configurations above.

For reasons of clarity, the drawings do not show all of the techniques used during rescue with a litter. Get training in these techniques before using the litter.

Warning: do not hoist the litter on the carry handles.

### A. Vertical litter

Evacuation is done using the head end attachment point. Adapted for narrow openings. Limits rockfall risk for the patient. Minimum patient comfort.

### B. Near-vertical litter

We advise you to use the STEF. Good comfort/bulk compromise.

### C. Horizontal litter

We advise you to use the STEF. Comfortable for the patient, but increases bulk and patient exposure to rockfall. May be ordered by the doctor.

### D. Tyrolean

## 10. Evacuation with the STEF

Each of the evacuation techniques with the STEF applies to the following two configurations:

- As of 2020: NEST and STEF.
- Before 2020: pre-2020 NEST and STEF for the pre-2020 NEST.

Be sure to follow the diagram for installing the STEF on the litter for each of the configurations above.

### a. Litter in horizontal position

To secure this position, clip the Sm'D to the SWIVEL.

### b. Changing from horizontal position to inclined position

The rescuer, attached to an independent rope, faces the litter. S/he detaches the Sm'D carabiner. The rescuer then positions his/her hands on the two straps (foot end) and tilts the litter, possibly by pushing on the litter with his/her feet. Warning: do not place your hands on the head end; you may not be able to control the rate of tilt, and there is a risk of pinching your hands.

### c. Changing from inclined position to horizontal position

The rescuer attaches his/her short lanyard to the ring on the STEF, which will serve as a fixed point to help guide the maneuver. The rescuer ensures that the lanyard is taut.

- The controller and counterweight lower the litter. It then tilts naturally.

- The rescuer clips the Sm'D into the SWIVEL to secure the horizontal position.

The controller and counterweight raise the litter. The rescuer removes his/her lanyard from the ring.

## 11. Transporting the litter

The litter can be stored in the carrying case (S062AA00). To transport the litter in a vehicle, you can remove the longitudinal and transverse stays to fold it.

## 12. Additional information

### When to retire your equipment

WARNING: an exceptional event can lead you to retire a product after only one use, depending on the type and intensity of usage and the environment of usage (harsh environments, marine environments, sharp edges, extreme temperatures, chemicals...).

A product must be retired when:

- It is over 10 years old and made of plastic or textiles.

- It has been subjected to a major fall or load.

- It fails to pass inspection. You have any doubt as to its reliability.

- You do not know its full usage history.

- When it becomes obsolete due to changes in legislation, standards, technique or incompatibility with other equipment...

Destroy these products to prevent further use.

### Icons:

**A. Lifetime: 10 years - B. Marking - C. Acceptable temperatures - D. Usage precautions - E. Cleaning/disinfection - F. Drying - G. Storage/transport - H. Modifications/repairs** (prohibited outside of Petzl facilities, except replacement parts) - **I. Questions/contact**

### 3-year guarantee

Against any material or manufacturing defect. Exclusions: normal wear and tear, oxidation, modifications or alterations, incorrect storage, poor maintenance, negligence, uses for which this product is not designed.

### Warning symbols

1. Situation presenting an imminent risk of serious injury or death. 2. Exposure to a potential risk of accident or injury. 3. Important information on the functioning or performance of your product. 4. Equipment incompatibility.

### Traceability and markings

a. Serial number - b. Year of manufacture - c. Month of manufacture - d. Batch number - e. Individual identifier - f. Read the Instructions for Use carefully - g.

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© Petzl